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Hold the white base with one hand and pull back the orange top (mouth piece) as shown. Be sure to open the top completely until the arrows line up with each other.

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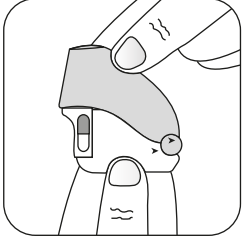
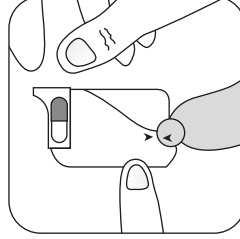
### Insert the Rotacap into the Rotacap chamber.

Place the transparent end on the bottom so you will be able to see when the medication is completely delivered.

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Close the mouth piece firmly until a click is heard which indicates proper locking of the Revolizer.

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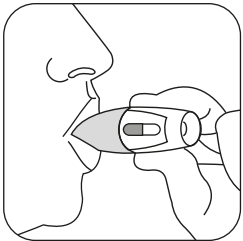
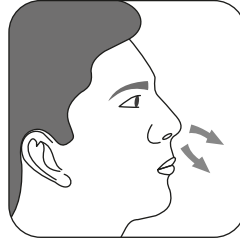
### Breathe out fully.

Breathe out fully before taking dose of Revolizer.

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Grip the mouthpiece between your teeth and seal your lips around it. Tilt your head slightly backwards. Breathe in through your mouth as deeply as you can. Remove the Revolizer from your mouth and hold your breath as long as possible then breathe out.

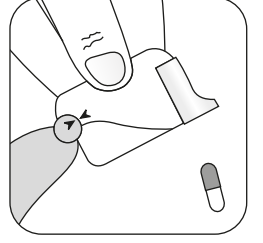
Note: If you are doing this properly you will hear the Rotacap shell vibrating inside the Revolizer.

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### 6 Open the Mouth Piece. Mouth Piece کو کھولیں۔

After every use, open the mouth piece of your Revolizer, till the arrows meet, then discard the used Rotacap.

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
Close the mouth piece firmly. Store Revolizer in convenient pouch which is provided with it. Every time use fresh Rotacap. Proceed with steps 1 through 7.

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### The Revolizer is robust and requires minimal cleaning and maintenance

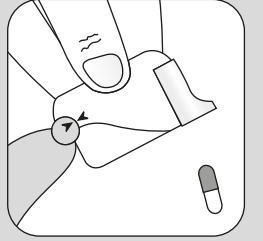- Wash the Revolizer once every 2 weeks.
- Open the mouth piece and remove any Rotacap shell that might be left in the chamber.
- Rinse the mouthpiece and chamber in running water.
- Remove excess water by tipping onto a paper towel and leave to air dry.
- Air drying takes about 6 hours so if possible wash the Revolizer and leave it to dry overnight.

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### General Care Tips for your Revolizer

- Do not push any cleaning cloth or instrument into the mouth piece of the Revolizer as it may cause damage.
- Only insert the Rotacap into the device just prior to use.
- Your Revolizer & Rotacap bottle come with a convenient pouch with space. After use, store both items together in a clean, dry place for easy access.
- Use only as directed by your doctor. If the recommended dose does not provide relief or symptoms or symptoms become worse, consult your doctor.
- It is recommended to replace your Revolizer every 6 months.

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